

Double Weighing Balance-ATL 3006



Features:

Operators put two buckets (with blood bags in each bucket) on both plates

Display shows weight difference and an indication lamp at the lighter side turns on.

Operators just add a few weights – chips (e. g. 1 gram each) into bucket until GOOD lamp turns on. The lamp turns on when weight difference becomes less than allowance

Accurate simple and very easy to use

Password Protection

Specification:

- Range of weight measurement: 0 -3000 grams/ml
- Weight difference to be displayed: 0-3000 grams/ml
- Weight difference for GOOD lamp” 2 gram (may vary upon request)
- Weight display panel: GOOD ,A,B
- Weight sensor: load cell
- Control: Power ZERO
- Accuracy: +/-2 gram
- Power consumption: 5 W
- Weighing Range: 2500 grams/ml
- Storage: -20 – 70⁰C (-4 – 150⁰F)
- Temp. operation: 0-40⁰C (32-104⁰F)